steril
sterile

SERVOCURA®

- REF 41148** Tag- und Nachtbeutel 1000 ml
mit T-Tap Ventil – 90 cm
S-UB 1000s 90 cm
- REF 41149** Tag- und Nachtbeutel 1000 ml
mit T-Tap Ventil – 130 cm
S-UB 1000s 130 cm
- REF 41148** Day and night bag 1000 ml
with T-Tap drain valve – 90 cm
S-UB 1000s 90 cm
- REF 41149** Day and night bag 1000 ml
with T-Tap drain valve – 130 cm
S-UB 1000s 130 cm

DE

EN

Orbisana Healthcare GmbH

Biberweg 24-26
53842 Troisdorf | Germany
Tel. +49 (0)2241 9322-0
Fax +49 (0)2241 9322-277
info@orbisana-hc.de
www.orbisana-hc.de

41270Version 2022-11-11

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SERVOCURA®

steril

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INSTRUCTIONS FOR USE

SERVOCURA® **sterile**

Day and night bag 1000 ml with T-Tap drain valve - 90 cm

S-UB 1000s 90 cm

Day and night bag 1000 ml with T-Tap drain valve - 130 cm

S-UB 1000s 130 cm

Symbols used

8

EN

1 Foreword

Dear customer,

We would like to thank you for your trust and the purchase of our quality product.

This instruction manual is intended for patients, relatives and medical staff.

The SERVOCURA® leg bag can be used at home and in a clinical setting. Independent use (without support from relatives or nursing staff, for example) is possible.

The SERVOCURA® leg bag should be used in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

Note

Read this instruction manual carefully. It contains important information on safe and proper use of the product. Keep this instruction manual for future reference.

SERVOCURA® day and night bags are assigned to batches for traceability purposes.

The batch number of each SERVOCURA® day and night bag is indicated on the instruction manual next to "LOT". This batch number allows each bag to be uniquely identified. If you have any questions or problems with your SERVOCURA® day and night bag, please contact your field representative or Servona GmbH, referencing this batch number.

1.1 Intended purpose

SERVOCURA® day and night bags are connected to urinary sheaths or urinary catheters and collect drained urine.

Note

The SERVOCURA® day and night bag is a sterile, disposable medical product and must not be reused, cleaned or reprocessed.

1.2 Indication

Urine drainage; treating moderate to severe incontinence.

1.3 Contraindication

Known sensitivity to the materials used.

1.4 Side effects

There are no known side effects provided the patient uses the product properly and observes the safety instructions.

41270Version 2022-11-11

2 Safety instructions

Safe and proper use of your urine bag is important for your health. Ignoring the safety instructions can have significant effects on your health or even cause life-threatening injuries.

Attention: The word “Attention” indicates a situation that can cause damage to your product if not avoided.

Caution: The word “Caution” indicates a situation that can result in minor or moderate injury if not avoided.

Please observe the following safety instructions:

Caution

Risk of infection: Using a urine bag for longer than intended can cause an infection.

- Replace the bag at least after 14 days.

Risk of infection: If the packaging or the product is damaged or contaminated, the product is no longer sterile. This may cause an infection.

- Do not use the product if the packaging or the product is damaged!

Risk of injury: If the urine bag is attached lower than your bladder, gravity causes the urine to flow down into the bag. If the urine bag is attached higher than your bladder, the urine cannot flow into the bag. This can cause a build-up in your urinary tract and bladder.

- Always attach the bag lower than your bladder so that the urine can flow out!

Risk of injury: If the urine bag is full and is not emptied in good time or if the tube is bent, this can cause a build-up in your urinary tract and bladder.

- Empty the urine bag in good time!
- Make sure that you do not bend the tube!

Attention

The product is sterilised with ethylene oxide. Resterilisation may damage the product!

- Do not reesterilise the product!

3 Description

Components:

- Urine drainage bag with anti-reflux valve, reverse side with fleece coating with bacteria- and liquid-tight bag venting system
- T-tap drain valve with silicone connector (to connect other urine bags)
- Flap for T-tap drain valve
- For SERVOCURA® day and night bag 90 cm (REF 41148): 90 cm tube with step connector with a needle-free sampling port
- For SERVOCURA® day and night bag 130 cm (REF 41148): 130 cm tube with step connector with a needle-free sampling port

Capacity: 1000 ml

Material: PVC

The tube can be shortened, a second connector is included.

Note

Volumes indicated on the bag are only guidelines!

4 Application

Wash the hands thoroughly before and after use.

A standard skin disinfectant can be used to disinfect the T-tap drain valve or the connection point between the urinary catheter and the bag.

4.1 Preparation

- Check the packaging for damage or impurities.
- Remove the bag from the sterile packaging. Make sure that you do not contaminate the connector.
- Check the bag for damage or impurities.
- Close the T-tap drain valve (if open).
- Insert the T-Tap drain valve into flap provided.
- Remove the protective cap on the tube connector.
- Disinfect the connection points between the catheter and the bag.

4.2 Connecting and securing the bag

To connect the bag to the catheter, hold the catheter securely and insert the connector of the bag into the Step connector of the catheter.

Secure the bag to the leg, with the fleece side to the skin, using a suitable fastening system (e.g. SERVOCURA® strap, art. 25226) or to the bed using a suitable fastening system (e.g. SERVOCURA® universal holder, art. 25275). Ensure that the bag is always below the bladder.

EN

4.3 Emptying the bag

To empty the bag, take the T-tap drain valve, hold the draining device down and open the T-tap drain valve. Once the bag is empty, close and disinfect the T-tap drain valve.

4.4 Removing / replacing the bag

To remove the bag, hold the catheter securely and pull the connector of the bag out of the step connector of the catheter. Then disinfect the connection points between the catheter and the bag.

The bag should be changed at least every 14 days.

4.5 Connecting another bag

For larger quantities of urine, e.g. at night, a bedside bag system can also be connected to a leg bag. To do so, connect the bedside drainage bag to the connection tube of the T-tap drain valve on the leg bag. Then open the T-tap drain valve on the leg bag.

To remove the bedside drainage bag, close the T-tap drain valve on the leg bag.

5 Period of use and disposal

The urine bag must not be used for longer than 14 days. The empty bag can be disposed of with household waste.

6 Disclaimer

In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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